

Horizon® HTP Engine Key

**Product
Bulletin**

No. 107

Summary

Horizon users with high throughput print engines upgrading to Version 1.7.0 software must install an "HTP Engine" key for maximum print speed. HTP Engine keys are available at no cost from Codonics Technical Support.

Bulletin Details

Horizon Software Versions 1.7.0 and later support high throughput and standard print engines. As of the date of this bulletin, all installed Horizon imagers contain high throughput print engines. 1.7.0 software requires the HTP Engine feature key to be installed for these imagers to print at maximum speed.

Imagers with high throughput print engines factory built with Horizon Software Versions 1.7.0 and later will automatically include the HTP Engine key.

You can check for the presence of the HTP Engine key by viewing an Imager Status test print or viewing the list of feature keys from the Feature Management->View Features menu.

You can obtain an HTP Engine feature key at no cost by contacting Codonics Technical Support at the phone numbers listed below. You will need to provide your imager serial number and your software license code, which can be viewed from an Imager Status test print or under the control panel Utilities->Imager Info screens. They are labeled "Imager S/N" and "License Code" respectively.

Get it all with just one call
1-800-444-1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice. Patents pending.

Copyright © 2000-2003 by Codonics, Inc. Printed in the U.S.A. Part No. 903-001-007 Rev.A.



We bring the future into focus

17991 Englewood Drive
Middleburg Heights, OH 44130 USA
(440) 243-1198
(440) 243-1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Limited KK
New Shibaura Bldg. F1
1-3-11, Shibaura
Minato-ku, Tokyo, 105-0023 JAPAN
Phone: 81-3-5730-2297
Fax: 81-3-5730-2295