

Codonics® Safe Label System® **Release Notes**

Operating Software

Version 3.3.1 SLS Release Notes

Summary

Version 3.3.1 software (SW) is a software release that is only supported on Codonics Safe Label System® (SLS) Point of Care Stations (PCS) with serial numbers that begin with 142C, 143C, 144C and 170C. Version 3.3.1 SW is NOT compatible with SLS PCS with serial numbers that begin with 140C and 141C. SLS PCS does not support software downgrades to previous software versions.

NOTE: The 3.3.0 Administration Tool (AT) software must be used with the 3.3.1 SLS PCS software.

The 3.3.0 AT works simultaneously with supported 3.3.x SLS PCSs.

This document provides information about Product Improvements (i.e., Perfective Updates) and Product Corrections (i.e., Corrective Updates) in 3.3.1 SLS PCS software.

WARNING: The use of label colors is intended only as an aid in the identification of drug groups and does not absolve the user from the duty of reading the label and correctly identifying the drug prior to use.

Product Improvements

- ◆ **SLS PCS now logs whether a Delayed Update timed out and auto-installed or the user clicked Install Now.** This will help in troubleshooting SLS PCS package updates.
- ◆ **SLS PCS now logs whether the SLS network time has been set using an NTP server.** This will help in troubleshooting issues with the NTP server.
- ◆ **The operating software includes security updates.** Refer to Codonics IT Resource Guide 900-653-016 for details.
- ◆ **The SLS supports new drug names.** The drug names are aprepitant, ascorbic acid, carbachol, epinephrine topical, factor 7a recombinant, ferric subsulfate, fluticasone, gentian violet, indomethacin, lidocaine endotracheal, and lidocaine jelly. Contact Codonics Technical Support to acquire these new drug names and for assistance installing them in your 3.3.0 AT.
- ◆ **The SLS supports new numbers.** The new number is 25.9. Contact Codonics Technical Support to acquire this new number and for assistance installing it in your 3.3.0 AT.

Product Corrections

- ◆ **SLS PCS drug audio sometimes failed to finish playing.** In rare occasions, when SLS announced the drug name or concentration part of it would fail to finish playing. This was corrected.
- ◆ **Previously clearing some error messages from the Utilities menu caused unintended behavior.** This issue was corrected.

Known Common Issues

This section details common issues with SLS PCSs that are likely to be experienced by most users. These issues are specific to the current release; previous issues can be found in previous Release Notes. None of the issues included in the document are considered to render the product unsafe or ineffective for clinical use.

- ◆ **Pressing the Load button does not always cut labels on SLS PCS.** When pressing the Load button, the SLS will feed media forward but does not cut it when the door is closed. Once a subsequent label is printed, the unit will eject the printed label attached to a blank label from pressing the Load button. When the Load button is pressed on SLS PCSs with serial numbers 140C and 141C, the unit will feed media forward. When the door is closed, it will cut a label.

Known Uncommon Issues

This section details uncommon issues with all SLS PCSs that are unlikely to be experienced by most users. These issues are specific to the current release; previous issues can be found in previous Release Notes. None of the issues included in the document are considered to render the product unsafe or ineffective for clinical use.

- ◆ **SLS PCS may start-up and display No Scanner.** If this occurs, reboot the SLS PCS.
- ◆ **The screen saver does not dismiss when touching the screen after setting date/time forward.** Plug a keyboard into the unit and press a key to dismiss the screensaver; the screen's operation will return to normal.
- ◆ **SLS PCS does not notify the user if a drug container's data matrix barcode has an invalid check digit.** All drug containers have to be verified in the AT and/or SLS PCS. Make sure you carefully review all the verification information before verifying the entry and printing a label.

Technical Support

If problems occur during software installation or operation, contact Codonics Technical Support at any time.

Phone: +1.440.243.1198
Email: support@codonics.com
Website: www.codonics.com

Get it all with just one call
1-800-444-1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.
Patents: www.codonics.com/ip/patents.

Copyright © 2010-2025 Codonics, Inc. Printed in the U.S.A. Part No. 901-243-045.02



17991 Englewood Drive
Cleveland, OH 44130 USA
(440) 243-1198
(440) 243-1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Trading Co, Ltd.
317 Xianxia Rd. Building B
Unit 1412
Changning Dist., Shanghai
P.R. China, 200051
86-21-62787701
86-21-62787719 Fax