

Codonics Virtua Ink Replacement

Technical Brief

Overview

This document explains how to properly replace the ink cartridges for the Codonics Virtua® Medical Disc Publisher 83C Series Recorder.


IMPORTANT: A firmware update is necessary in order to continue supporting ink for this legacy series Recorder (serial number beginning with 83C). See the Virtua 83C Recorder Firmware Upgrade Technical Brief (901-595-001) for installation instructions.

NOTE: It is strongly recommended that you keep spare ink cartridges on site to ensure that there is no system downtime if an ink cartridge must be replaced.

Replacing the Ink Cartridge


1. Open the front Cover.

CAUTION: Do not touch a patient while accessing Virtua's internal components that are under the front cover.

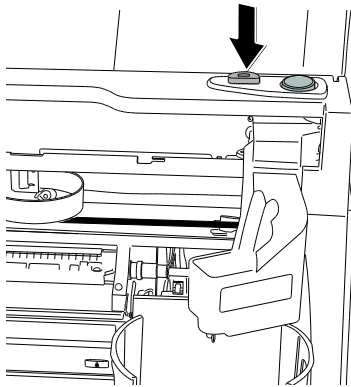


2. Press the Ink Cartridge Load button.


NOTE: Do not open the print head latch when replacing an ink cartridge.

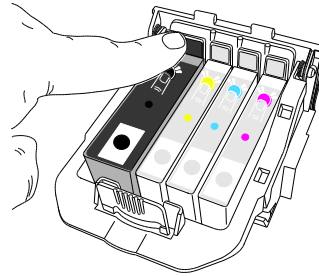


3. Press the colored tab on the back of the print head that corresponds to the used ink cartridge. The ink cartridge pops up slightly.

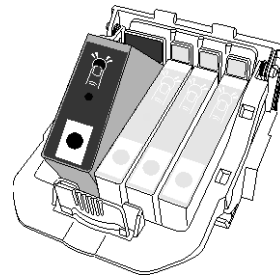

4. Remove the used ink cartridge.

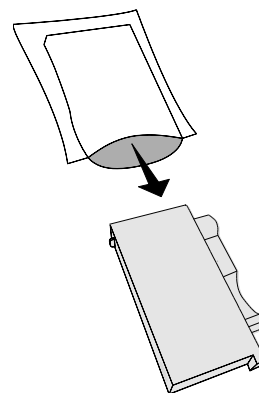


5. Remove the new ink cartridge from its packaging.

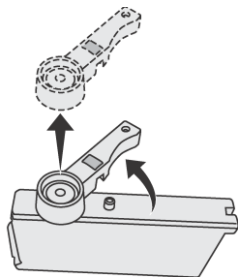


6. Remove the ink protective cap from the ink cartridge.

NOTE: Install the ink cartridge immediately after removing the protective cap to prevent the ink from drying out.

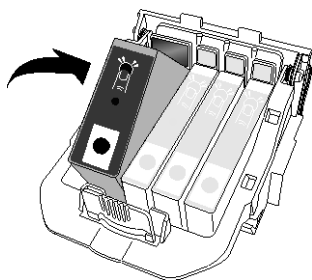

7. Place the ink cartridge into its corresponding colored slot.

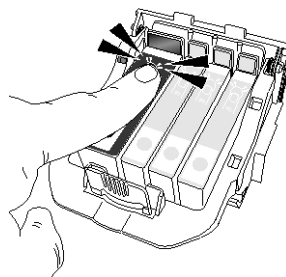

8. Press the back of the ink cartridge down until you hear a click.

NOTE: For the printer to operate, all four ink cartridges must be installed.

9. Close the front cover.
10. Wait approximately two minutes while the system performs a maintenance cycle on the ink system.

NOTE: Do not shut down the device while the system is performing the ink system maintenance cycle.

Technical Support

If problems occur when replacing an ink cartridge, contact Codonics Technical Support.

Phone: +1.440.243.1198
Email: support@codonics.com
Website: www.codonics.com

Get it all with just one call
800.444.1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.
Patents: www.codonics.com/ip/patents.

Copyright © 2026 Codonics, Inc. Printed in the U.S.A. Part No. 901-594-002.01.



17991 Englewood Drive
Middleburg Heights, OH
44130 USA
+1.440.243.1198
+1.440.243.1334 Fax
Email: info@codonics.com
www.codonics.com