

# Codonics® SLS 500i® Operating Software

## Release Notes

### Version 2.1.2 SLS 500i Release Notes

#### Summary

Version 2.1.2 software is a software release for the Codonics Safe Label System® 500i (SLS) Point of Care Station (PCS) and replaces 2.1.0 and previous software versions. SLS PCS does not support software downgrades to previous software versions.

**NOTE:** The 2.1.0 Administration Tool (AT) software must be used with the 2.1.2 SLS PCS software.

The 2.1.0 AT works with all 2.1.x SLS PCSs simultaneously.

As of this release, SLS software version 2.1.2 is considered the most current, stable, safe and effective version for **previous** SLS PCS. SLS software version 2.1.3 is considered the most current, stable, safe and effective version for the **updated** SLS PCS. Codonics recommends and encourages updating to these versions.

**NOTE:** The serial number for the **previous** SLS PCS with the new printer begins with the number 140C or 141C.

The **updated** SLS PCS serial number began with either the number 142C.


**NOTE:** 2.1.2 SLS SW will not install on **updated** versions of SLS PCS (i.e., those starting with 142C).

This document provides information about Product Corrections (i.e., Corrective Updates). The SLS PCS User's Manual version 1.3.0 and User's Manual Addendum 1.8.0 provide further information about SLS PCS.

**WARNING:** The use of label colors is intended only as an aid in the identification of drug groups and does not absolve the user from the duty of reading the label and correctly identifying the drug prior to use.

### Product Corrections

- ◆ **Multi-package updates can now be installed on SLS PCS.** SLS Product Bulletin 406 identified an issue where if a multi-package update was installed on SLS PCS, the wrong drug could be displayed on SLS. This issue no longer occurs. Refer to SLS Release Notes 901-245-014 and SLS AT File Manager Technical Brief 901-276-001 for details on multi-package updates.

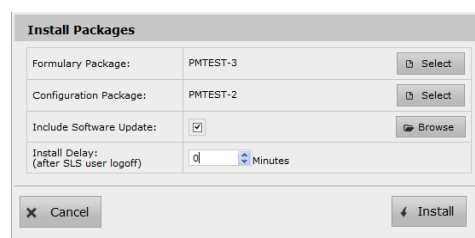

- ◆ **Previously installing a 2.1.0 APP Package on a higher version of SLS SW (e.g., 2.1.1) using a USB drive would hang-up during installation.** Now, the SLS will not allow a 2.1.0 APP package to be installed using a USB drive on a SLS PCS that has a higher version (i.e. 2.1.2 or 2.1.3) of SLS SW. It will display the 2.1.0 APP package as INVALID.

### Technical Support

If problems occur during software installation or operation, contact Codonics Technical Support at any time.

Phone: +1.440.243.1198

Email: support@codonics.com

Website: www.codonics.com

*Get it all with just one call*  
**1-800-444-1198**

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice. Patents pending.

Copyright © 2010-2016 Codonics, Inc. Printed in the U.S.A. Part No. 901-243-022.02



17991 Englewood Drive Cleveland, OH 44130 USA (440) 243-1198 (440) 243-1334 Fax Email info@codonics.com www.codonics.com	Codonics Trading Co, Ltd. 317 Xianxia Rd. Building B Unit 1412 Changning Dist., Shanghai P.R. China, 200051 86-21-62787701 86-21-62787719 Fax	Codonics Limited KK AQUACITY 9F, 4-16-23, Shibaura Minato-ku, Tokyo, 108-0023 JAPAN 81-3-5730-2297 81-3-5730-2295 Fax
---	---	---