

Codonics Container Labeling System Barcode Scanner Configuration

Tech Brief

Overview

The Codonics Container Labeling System™ (CLS) includes a Motorola® Symbol™ DS9808 barcode scanner.

The Symbol barcode scanner is not configured as shipped. You must perform the procedure in “Configuring the Symbol 9808 Barcode Scanner” to configure it.

If you are using another type of barcode scanner, refer to “Parameters to Enable for Other Scanner Types” for a list of the parameters that must be enabled to properly configure the scanner for use with CLS.

Configuring the Symbol 9808 Barcode Scanner

Scan the 2D barcode below to configure the Symbol 9808 barcode scanner for use with your CLS.



After successfully scanning this barcode, you will hear an audible beep, followed by an audible sound that indicates that the scanner is restarting.

Parameters to Enable for Other Scanner Types

The following parameters must be enabled to properly configure a scanner for use with CLS.

Parameter	State
UPC-A	Enabled
UPC-E	Enabled
UPC-E1	Enabled
EAN-8/JAN 8	Enabled
EAN-13/JAN 13	Enabled
Code 128	Enabled
GS1-128 (formerly UCC/EAN-128)	Enabled
ISBT 128	Enabled
Check ISBT Table	Enabled
Code 39	Enabled
Convert Code 39 to Code 32 (Italian Pharmacy Code)	Enabled
Code 32 Prefix	Enabled
Interleaved 2 of 5 (ITF)	Enabled
GS1 DataBar-14	Enabled
GS1 DataBar Limited	Enabled
GS1 DataBar Expanded	Enabled
Data Matrix	Enabled
Inverse 1D	Inverse Autodetect
Data Matrix Inverse	Inverse Autodetect
Presentation Performance Mode	Traditional
Carriage Return	Enabled
Caps Lock Override	Enabled

Technical Support

Please contact Codonics Technical Support at any time.

Phone: +1 (440) 243-1198
Email: support@codonics.com
Website: www.codonics.com

Get it all with just one call
1.800.444.1198

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice.

Copyright © 2012 Codonics, Inc. Printed in the U.S.A. Part No. 901-448-001.02.



We bring the future into focus

17991 Englewood Drive
Cleveland, OH 44130 USA
+1 (440) 243-1198
+1 (440) 243-1334 Fax
Email info@codonics.com
www.codonics.com

Codonics Limited KK
New Shibaura Bldg. F1
1-3-11, Shibaura
Minato-ku, Tokyo, 105-0023 JAPAN
Phone: 81-3-5730-2297
Fax: 81-3-5730-2295