

# Codonics® Safe Label System® **Release Notes**

## Operating Software

### Version 2.3.2 SLS Release Notes

#### Summary

Version 2.3.2 software is a software release for the Codonics Safe Label System® (SLS) Point of Care Station (PCS). SLS PCS does not support software downgrades to previous software versions.

**NOTE:** The 2.3.0 Administration Tool (AT) software must be used with the 2.3.2 SLS PCS software.

The 2.3.0 AT works with all 2.3.x SLS PCSs simultaneously.

This document provides information about Product Improvements (i.e., Perfective Updates) in 2.3.2 SLS PCS software.

**WARNING:** The use of label colors is intended only as an aid in the identification of drug groups and does not absolve the user from the duty of reading the label and correctly identifying the drug prior to use.

#### Product Improvements

- ◆ **Software support was included for future hardware capability.** There is no effect on any of the current SLS hardware platforms.

#### Known Common Issues

This section details common issues with SLS PCSs starting with serial numbers 142C, 143C and 144C that are likely to be experienced by most users. These issues are specific to the current release; previous issues can be found in previous Release Notes. None of the issues included in the document are considered to render the product unsafe or ineffective for clinical use.

- ◆ **Pressing the Load button does not cut labels on the updated SLS PCS.** When pressing the Load button, the unit will feed media forward but does not cut it when the door is closed. Once a subsequent syringe label is printed, the unit will eject the printed syringe label attached to a blank label from pressing the Load button. When the Load button is pressed on SLS PCSs with serial numbers 140C and 141C, the unit will feed media forward. When the door is closed, it will cut a label.

#### Known Uncommon Issues

This section details uncommon issues with updated SLS PCSs that are unlikely to be experienced by most users. These issues are specific to the current release; previous issues can be found in previous Release Notes. None of the issues included in the document are considered to render the product unsafe or ineffective for clinical use.

- ◆ **The Batch Mode button is displayed on SLS PCS when Copy Mode configuration is set and installed.** If a Batch/Copy Mode key is installed after a Copy Mode configuration is installed on SLS, then the Batch Mode button will be displayed. Feature keys should be installed before installing Copy Mode configuration or reinstall the Copy Mode configuration package after the Batch/Copy Mode key is installed.
- ◆ **The screen saver does not dismiss when touching the screen after setting date/time forward.** Plug a keyboard into the unit and press a key to dismiss the screensaver; the screen's operation will return to normal.
- ◆ **User packages are not added to multiple update packages when using the Combine Packages feature.** Multiple update packages that contain the User package, and are installed over the network do not install correctly. This issue only occurs when trying to use the Combine Packages feature and installing a multiple update via USB. If performing a USB install and requiring a User package update, you should create a multiple update package with formulary, configuration, and/or software packages and a separate User package. Include both the multiple update package and the separate User package on the USB and install them separately on SLS PCS.

#### Technical Support

If problems occur during software installation or operation, contact Codonics Technical Support at any time.

Phone: +1.440.243.1198

Email: support@codonics.com

Website: www.codonics.com

*Get it all with just one call*  
**1-800-444-1198**

All registered and unregistered trademarks are the property of their respective owners. Specifications subject to change without notice. Patents: [www.codonics.com/ip/patents](http://www.codonics.com/ip/patents).

Copyright © 2010-2018 Codonics, Inc. Printed in the U.S.A. Part No. 901-243-030.01



17991 Englewood Drive Cleveland, OH 44130 USA (440) 243-1198 (440) 243-1334 Fax Email <a href="mailto:info@codonics.com">info@codonics.com</a> <a href="http://www.codonics.com">www.codonics.com</a>	Codonics Trading Co, Ltd. 317 Xianxia Rd. Building B Unit 1412 Changning Dist., Shanghai P.R. China, 200051 86-21-62787701 86-21-62787719 Fax	Codonics Limited KK AQUACITY 9F, 4-16-23, Shibaura Minato-ku, Tokyo, 108-0023 JAPAN 81-3-5730-2297 81-3-5730-2295 Fax
----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------------------------------	-----------------------------------------------------------------------------------------------------------------------------------------